

STEP

1

Preparing Kit Items for Prostate Biopsy:

It is recommended to wear gloves

1. Remove BxBoards from Mylar bags (do not discard).
2. Remove BxBoards from clear containers (do not discard).
3. Write patient name and DOB on the back of each BxBoard.
4. Remove clear cover right before biopsy to retain moisture.

STEP

2

Outliers/Extra Cores (if applicable):

For additional cores taken from the 12 traditional sites or from outlier areas

1. Prepare BxBoard labeled "X1" by following steps 1-4 in "Preparing Kit Items".
2. Place up to one outlier/extra core into each lane.
3. Immediately write the site initials in the space provided above lane (i.e. RB, LLA, etc.).
4. Replace the clear cover.

STEP

3

Loading Tissue:

It is recommended the physician pass the biopsy gun to an assistant

1. Place one core into each site-specific lane.
2. Once all cores are loaded, replace the clear cover.

STEP

4

Mailing Instructions:

1. Ensure all BxBoards include patient name/DOB.
2. Seal BxBoards back into their clear containers and mylar bags.
3. Place all BxBoards into the collection kit.
4. Include patient demographic/insurance information.
5. Place kit into the provided shipping mailer.

Please note: It is the clinic's responsibility to ensure that the shipment containing biopsy specimens is in adherence with parcel courier guidelines and all applicable local, state, and federal shipping regulations.

If you have any questions about the BxBoard or your shipment call Lumea support at 844-960-3658
All used BxBoards must be labeled with patient name & DOB